

# Pedicraft Bassinets

## Operations Manual

**For technical assistance call 800-223-7649**

Pedicraft, Inc.  
4134 St. Augustine Road  
Jacksonville FL 32207

800-223-7649  
904-396-9627  
fax 904-346-3947

[www.pedicraft.com](http://www.pedicraft.com)



Rev. 5 011317

## Safety Guidelines

- **WARNING:** To avoid tipping hazards of this device, make sure that the basket or bed component sits firmly in the base and that all doors, drawers and casters are secure.
  - Lock casters when not in transit.
  - Optional Conductive Casters: Contact with oil should be avoided. Resistance of electrostatic conductive rubber tread compounds may increase with age, service and contact with lubricants. Some waxes, polishes and cleaning materials used for floor maintenance may adversely affect the resistance of the material. Since the manufacturer has no control over these conditions, no guarantee of the continuing level of conductivity is made. It is recommended that the resistance of the conductive assembly be initially tested, prior to use, and thereafter tested routinely at intervals of not more than one month, as a check on resistance value.
- 

## Cleaning Instructions

FAILURE TO FOLLOW CLEANING INSTRUCTIONS MAY VOID WARRANTY

Use a damp sponge or soft cloth with mild soap or detergent. **Be sure to follow manufacturer's directions for diluting cleaning solutions.** Rinse with clean water and dry.

### **Additional Cleaning Instructions** for Bassinet Carts

You may also use stainless steel cleaners such as *Stainless Steel Magic*. Follow manufacturer's directions.

### **Additional Cleaning Instructions** for Bassinet Mattress Pad

**To disinfect:** Remove mattress pad from Bassinet Basket. Use disinfectants recommended by Herculite, either phenolic type or quaternary type. Iodophor type disinfectants are not recommended as staining may result.

## **Additional Cleaning Instructions for Acrylic Bassinet Basket**

**Do not** expose to temperatures higher than 140° F.

**Cleaning Instructions:** Apply mild detergent solution with soft, clean cloth and allow the surface to dry.

**Disinfecting Instructions:** Dilute an antiseptic or hospital concentrate in the amount recommended on the label for general disinfection. Tested products that are compatible are hydrogen peroxide disinfectant spray and disinfecting bleach spray. Spray solution and use a soft, clean cloth to wipe the basket and allow the surface to dry.

**WARNING:** Do not allow concentrated disinfectant, surgical or methylated spirits, quaternary ammonia compounds, any liquid containing isopropyl alcohol or any other solvents to come in contact with the bassinet basket. Do not use a cleaner or disinfectant if the label advises against use on acrylic plastics.

---

## **Bassinet Latch Repair Instructions**

Models #18215, 18218, 18250 - order repair kit # 91805

Model #18270 - order repair kit #91806

1. Open Drawer.
2. Push Knob to right to position screw at opening inside drawer.
3. Unscrew Knob and remove both Knob and screw.
4. Pull black plastic Latch Slide from left. Remove Latch Slide and Spring.
5. (If Spring does not come out with the Slide, remove drawer, turn sideways and shake drawer until Spring falls out.)
6. Insert new Spring into slot at end of new Latch Slide.
7. Position Slide so that angled edge faces in the direction the drawer will close.
8. Insert Slide and Spring into opening.
9. Reinsert screw from inside Drawer and attach Knob. Tighten securely. (Lok-Tite applied to screw before attaching Knob is recommended.)

## Warranty

Pedicraft, Inc. guarantees Bassinets to be free of defects in workmanship or materials for a period of three years from the date of shipment to the purchaser. In case any purchaser finds a defect in workmanship or materials, promptly notify Pedicraft at 800-223-7649.

The purchaser must return the article as per instructions and Pedicraft will replace, repair or refund the purchase price, at its sole discretion. All replaced or repaired items will be shipped freight prepaid from the factory to the purchaser. Any extra costs resulting from failure to obtain prior shipping instructions from Pedicraft will be borne by the customer.

Any changes or modifications of the equipment by the purchaser without the express written permission of Pedicraft, Inc., will void this warranty. Failure to follow cleaning instructions may void warranty.



*Keeping Babies and Children Safe  
and Comfortable for over 40 years*

800-223-7649