

## Care and Maintenance of Pedicraft Bassinet Baskets

### Safety Guidelines

- **WARNING:** To avoid tipping hazards of this device, make sure that the basket or bed component sits firmly in the base and that all doors, drawers and casters are secure.
- Lock casters when not in transit.

### Cleaning Instructions for Acrylic Bassinet Basket

**Do not** expose to temperatures higher than 140° F.

**Cleaning Instructions:** Apply mild detergent solution with soft, clean cloth and allow the surface to dry.

**Disinfecting Instructions:** Dilute an antiseptic or hospital concentrate in the amount recommended on the label for general disinfection. Tested products that are compatible are hydrogen peroxide disinfectant spray and disinfecting bleach spray. Spray solution and use a soft, clean cloth to wipe the basket and allow the surface to dry.

**WARNING:** Do not allow concentrated disinfectant, surgical or methylated spirits, quaternary ammonia compounds, any liquid containing isopropyl alcohol or any other solvents to come in contact with the bassinet basket. Do not use a cleaner or disinfectant if the label advises against use on acrylic plastics.

Rev. 3 011317

